

# Smile...its only LASIK

1

DOESN'T THIS LOOK LIKE A SPACE SHIP?



2




THE INCREDIBLE VISUMAX PEDESTAL BED CAN LOCK AT 45 DEGREES (YOU MUST TURN ON THE FEMTO FOR CONTROL)

**How to make your new VISUMAX work well with your NIDEK or  
How to make your new NIDEK work well with your VISUMAX**

3


YOU CAN USE THE 45 DEGREE POSITION WITH A MICROSCOPE

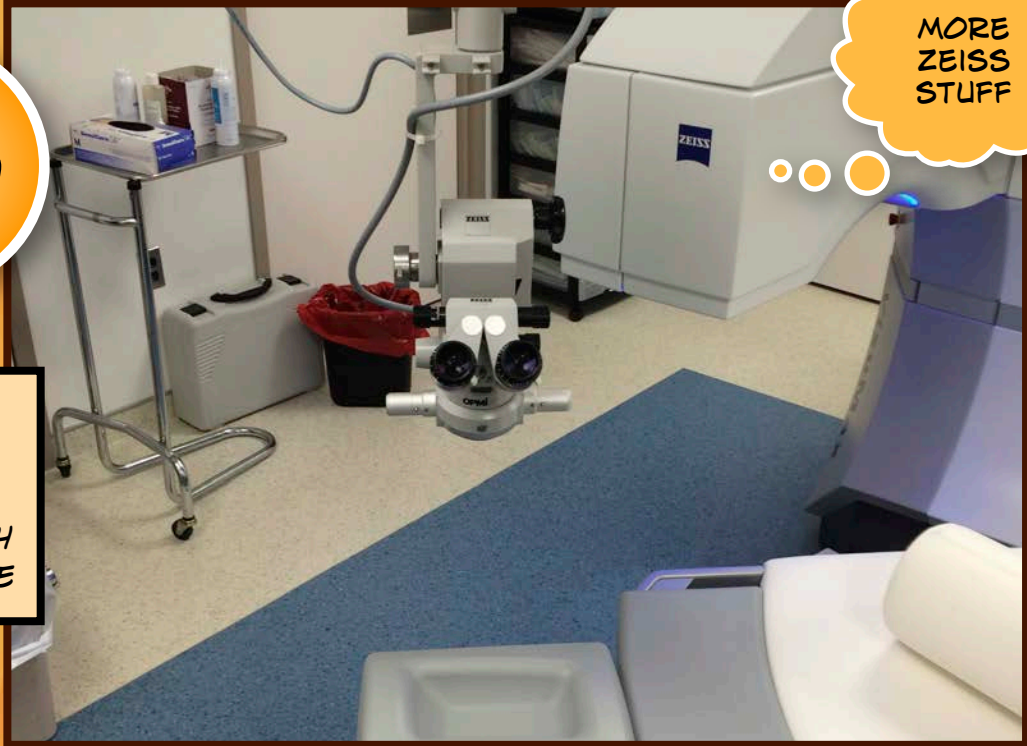

4




IF YOU SET IT UP JUST RIGHT THE VISUMAX BED WILL CONTINUE TO 90 DEGREES AND LOCK IN FOR USE WITH YOUR NIDEK.  
DON'T ASK NIDEK TO HELP YOU....THEY ARE WAY TOO UPTIGHT  
YOU HAVE TO REMOVE ONE OF THE FOUR WHEELS--- OH MY!

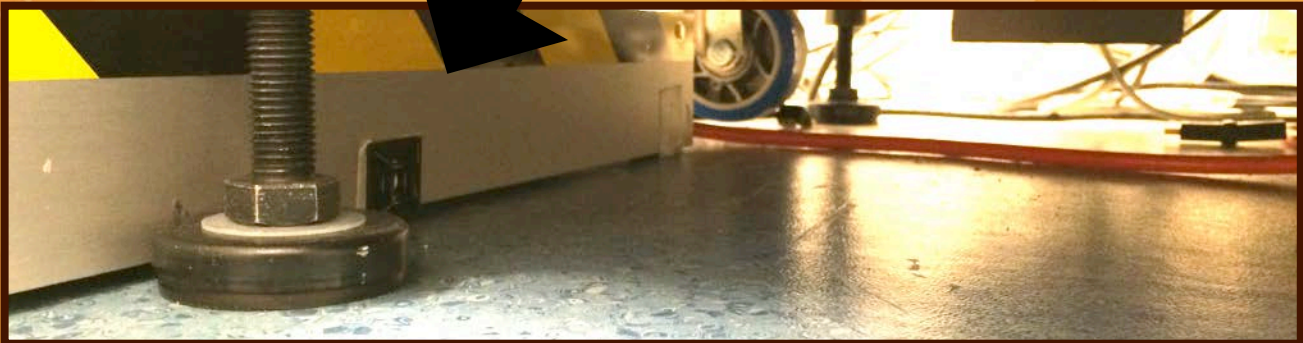

5

THIS IS THE PROPER POSITION OF THE NIDEK STRUT

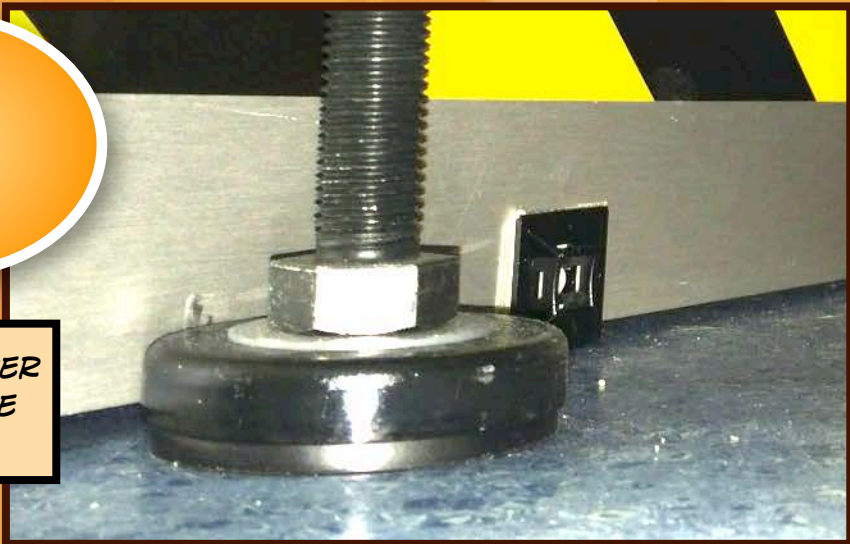


6



THERE IS A MILD OVERHANG

PERFECT CLEARANCE, COMPLETE CONTROL AND A PATIENT BED THAT WORKS AS WELL FOR THE NIDEK AS IT DOES FOR THE VISUMAX

THIS BOLT RETRACTS THE STRUT

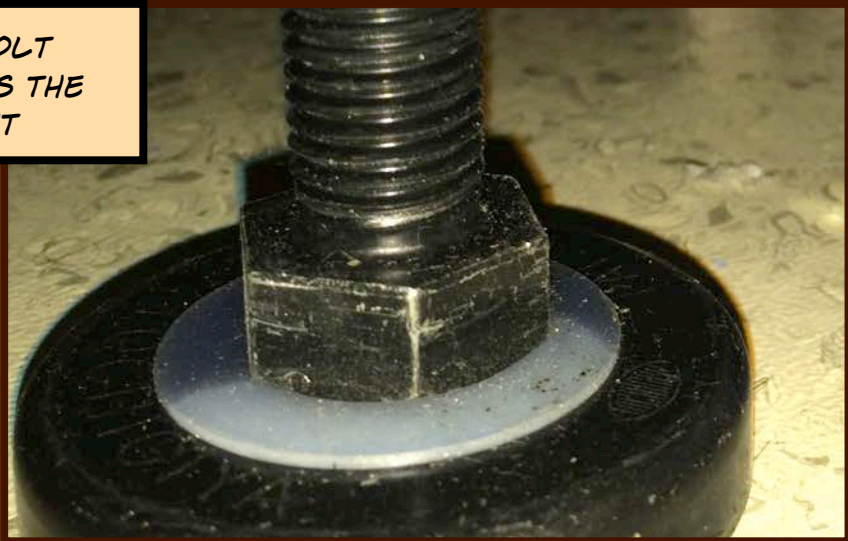

7

LIFT IT UP,  
TAKE OFF THE  
WHEEL, LOWER IT  
ONTO THE WHEELS,  
MOVE THE LASER,  
EXTEND THE STRUT  
AND YOU ARE DONE

8



STORE THE WHEEL AND  
SCREWS IN CASE YOU NEED  
TO MOVE YOUR LASER.  
QUESTIONS JUST ASK  
[JSLEWIS@ME.COM](mailto:JSLEWIS@ME.COM)

**Use a wrench to lift the NIDEK off its wheels by turning the struts. The struts will extend very easily. Remove the wheel nearest the VISUMAX. Next, retracted the struts so the NIDEK rests on 3 wheels. My wife pushed down on the opposite edge of the NIDEK for balance and I moved the machine about 6 inches into the position shown above. I extended the struts again and lifted the NIDEK off its wheels and into its final position. Checked the NIDEK with a level and was done. Five minutes max.**